



Pfizer: Committed to meeting your individual treatment needs

Ordering your Sleeve is free and easy.
For any questions about the Sleeve or to order:



Call **0800 ENBREL**
(0800 362 735)

Please contact your rheumatologist or biologics nurse if you have any questions about your treatment with **ENBREL**.


References: 1. Pfizer data on file. Auto-injector specifications. October 2018. 2. The Sleeve for ENBREL Auto-injector Instructions for Use leaflet.



Pfizer New Zealand, Auckland.
Pfizer Medical Information: 0800 736 363. www.pfizer.co.nz.
PP-ENB-NZL-0143-December 2019. NA 11457. PZR0508.
Enbrel® is a registered trademark. © Pfizer 2019.




YOUR GUIDE TO USING THE SLEEVE FOR ENBREL AUTO-INJECTOR

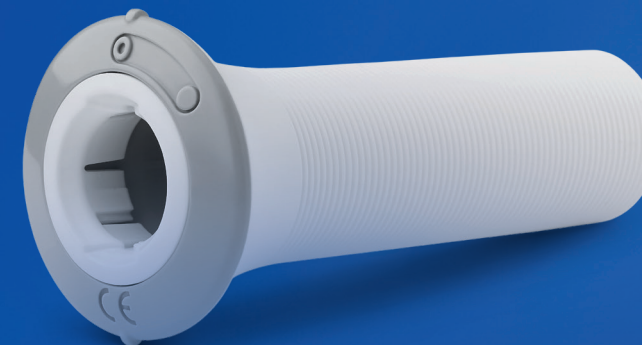

Designed to support your individual needs
when using the **ENBREL** Auto-injector.

Please ensure that you have read and fully understood the Instructions for Use for BOTH your **ENBREL** Auto-injector and the Sleeve for ENBREL Auto-injector **BEFORE** administering an injection. Please also refer to the **ENBREL** Consumer Medicine Information.

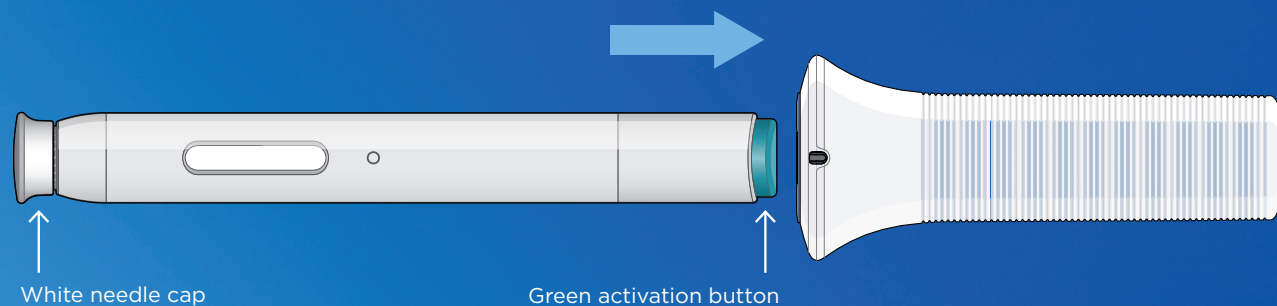

No need to press the button with thumb¹




Larger surface area to grip and hold²


Instructions for storage and cleaning

- The Sleeve may be **reused by a single patient for 2 years**
- It should be **stored in a clean dry place at room temperature**, out of reach and sight of children
- The Sleeve **may be washed with warm water** between uses
- **Discard or replace** if the Sleeve shows signs of wear


How to use the **Sleeve** in four simple steps¹

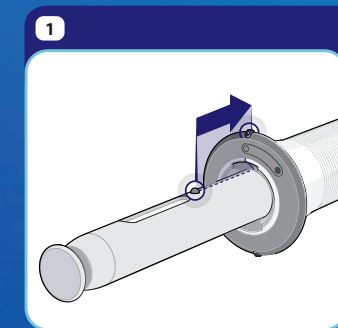


Insert the Auto-injector all the way into the Sleeve keeping the dots aligned.

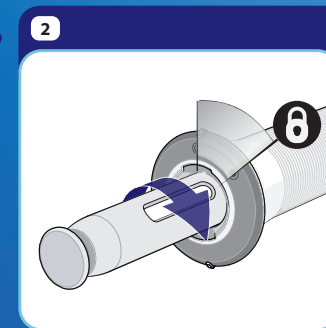

Twist the Auto-injector to lock it into the Sleeve.

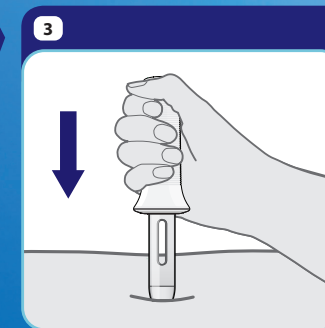

Remove the needle cap and position the Auto-injector against the skin as instructed in your Auto-injector Package Leaflet. Push the Sleeve firmly down to activate your injection.

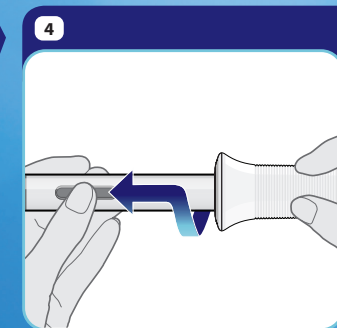

Twist the Auto-injector to unlock and remove from Sleeve.



If you have any questions about the Sleeve for ENBREL Auto-injector, call 0800 ENBREL (362 735)